

<h1>Installation of Lid Switch</h1>	ColoQuick	Thaw, Pasteur Basic2		




**Step 1: Tools Needed**  
**Items:** 7mm socket, Chanel locks, Screw drivers



**Step 2: Remove Switch**  
 Remove the old switch by removing the four mounting screws, **keep these screws**



**Step 3: Remove wire cover**  
 Remove the screw holding the wire cover on.

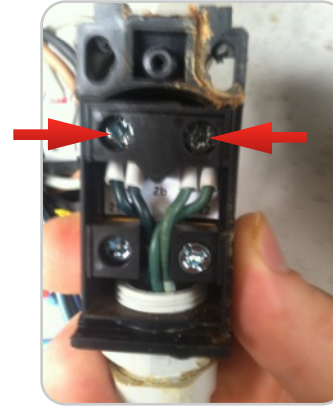

**Step 4: Removing wires.**  
 Loosen the two screws holding the wires in the switch, and remove the wires.



**Step 5: Remove wire retainer**  
 Unscrew both the cap and wire retainer from the lid switch.



**Step 6: Remove mounting screws**  
 Remove the four screws from the top of the new lid switch, these are too short and will be replaced with the **old screws**



**Step 7: Remove plug**  
 Remove the plug from the bottom of the new switch. Remove the wire cover (like in step 3).



**Step 8: Re-assembly**  
**Items:** Insert the wires into the new switch making sure wires of the same colors are together. Replace wire retainer and cover. Do not over tighten wire tainer



**Step 9: Install**  
**Items:** In stall the new switch using the four old screws.

**When working on any wiring in this unit make sure to turn it off and un-plug it!**