



Stainless steel tube feeder

This stainless steel tube feeder is a hygienic and efficient way to feed your calves. The design of this tube feeder makes it simple to hold and control flow with the flip of your thumb. It is easy to take apart and clean and the following steps will demonstrate how to do so.



We have a brush kit available for purchase to clean the stainless steel tube. If you have any further questions, call 715-318-0897.



Steps for taking apart tube feeder:

Step 1:

Separate the green handle by pulling it down the stainless steel tube.



Step 2:

Remove clamp from the neck of the white plastic piece.





Step 3:


Remove screws (3) and metal ring. When removing the metal ring make sure the notches on the white plastic piece lines up with that of the metal ring.



Step 4:

Separate white plastic head. This will expose the silicone hose and the flow control mechanism.

Replace silicone hose on a regular basis to ensure proper hygiene.

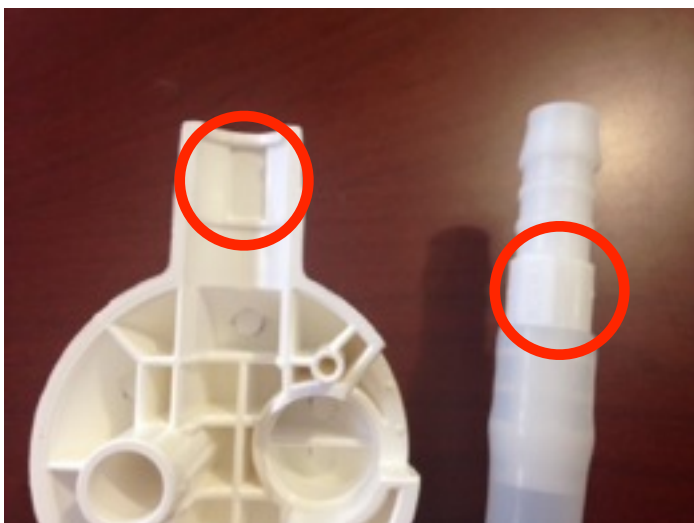

Step 5:

When installing the new silicone hose, make sure that the rectangle on the barbed fitting is fitted to the rectangle on the neck of the white plastic piece. This will allow the two white pieces to fit properly when you put the screws back.



If you need replacements,
call 715-318-0897.